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**Instructions of use O7 active® PRO gel**

**What is O7 active® PRO gel?**

O7 active® PRO gel is a product with a patented active oxygen technology. This technology becomes active at the moment of contact with the skin or mucous membranes.

**How does the O7 active® PRO gel work?**

O7 active® PRO gel contains the patented Ardox-X® technology which becomes active at the moment of contact with the skin or mucous membranes. The technology is based on a stabilized hydrocarbon-oxoborate complex that generates a non-radical form of oxygen under the influence of saliva. The hydrocarbon-oxoborate complex functions as a matrix that releases active oxygen on the area that has to be treated. The oxygen-enriched micro-environment has a proven positive effect on the (infected) wound treatment, e.g. infected pockets.

**For what purpose can the O7 active® PRO gel be used?**

O7 active® PRO gel is used for the treatment of periodontitis and peri-implantitis.

**In which cases can the O7 active® PRO gel be used?**

O7 active® PRO gel is used in dentistry in case of pockets with a minimum depth of 5 mm and peri-implantitis. O7 active® PRO gel is used in dentistry in case of peri-implantitis.

**In which case the O7 active® PRO gel can't be used?**

O7 active® PRO gel cannot be used in combination with metal implants or metal prosthesis, with the exception of titanium. Before using O7 active® PRO gel, first remove the prosthesis. Due to the fact that the O7 active® PRO gel reacts with metals, only plastic needles are suitable for the application of the O7 active® PRO gel.

**How is the O7 active® PRO gel used?**

Dosage

The dosage depends on the type of pocket and the depth of the pocket. The maximum dose per treatment is 3ml.

How is the O7 active® PRO gel used in dentistry in case of A pocket with a minimum depth of 5 mm?

- Place a plastic needle under the pocket and push the O7 active® PRO gel very slow out of the syringe until the O7 active® PRO gel that comes out of the pocket looks clean. Immediately after treatment of the pocket, the patient can have a warm and / or painful feeling for a short period of time, which should disappear almost immediately.
- Leave the O7 active® PRO gel in situ for 5 minutes.
- Then clean the pocket with physiological saline solution.
- Place a plastic needle under the pocket and push the O7 active® PRO gel out of the syringe very slowly until the O7 active® PRO gel that comes out of the pocket looks clean. Then leave the O7 active® PRO gel in the pocket.
- In exceptional cases, plugging of the needle may occur when capillary needles are used. In that case, use a new needle or a needle with a larger diameter.

**When is extra caution required with the O7 active® PRO gel?**

Possible reactions

O7 active® PRO gel will increase the pressure locally. The patient may feel pain, which must disappear quickly. This pain can also be felt in the head. These symptoms are the result of the local pressure increase. Treat furcations with special care.

What to do if the pain doesn't disappear?

Fill a syringe with physiological saline solution and rinse the pocket until the pain disappears.

How to use in dentistry in case of peri-implantitis?

Place a plastic needle under the pocket and push the O7 active® PRO gel out of the syringe very slowly until the O7 active® PRO gel that comes out of the pocket looks clean. Then leave the O7 active® PRO gel in the pocket.

What should you pay attention to during a pregnancy?

O7 active® PRO gel may, to the best of our knowledge, be used without risk to the embryo during pregnancy and / or lactation, according to the instructions. In general, you should use as few medicines as possible during the first trimester of pregnancy. O7 active® PRO gel is not a medicine. Nevertheless, restraint with the use during this period is still advised.

After periodontal surgery during which the O7 active® PRO gel has been used, the patient should continue with O7 active® PRO mouthwash (REF: O770) according to the manufacturer's instructions.

**Additional information O7 active® PRO gel.**

The critical ingredient in the O7 active® PRO gel is the Ardox-X® technology; based on an oxygen donating complex.

**Appearance and packaging of the O7 active® PRO gel.**

Appearance: O7 active® PRO gel is an opaque, white gel.

Packaging: O7 active® PRO gel is packed in plastic syringes with a content of 1 ml. One outer packaging contains 3 syringes.

These instructions for use were last revised in 02-2018.

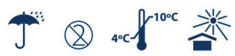
Ardox Healthcare BV,  
Kastelleinkenkampweg 9 C  
5222 AX 's-Hertogenbosch  
The Netherlands

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Distributed by: Sunstar Oral Company Int. B.V.