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GB Instructions for Use

What is OXYSAFE Gel Professional?

OXYSAFE Gel Professional is a product with a patented active oxygen technology. This technology becomes active at the moment of contact with the skin or mucous membrane.

How does the OXYSAFE Gel Professional work?

OXYSAFE Gel Professional contains the patented Ardoz-X™ technology which becomes active at the moment of contact with the skin or mucous membrane. This technology is based on a stabilised hydrocarbon-oxoborate complex in which a non radical species of oxygen is released under the influence of saliva. The hydrocarbon-oxoborate complex functions as a matrix that releases active oxygen on the area to be treated. The oxygen-enriched microenvironment has a proven positive effect on the (infected) wound treatment, e.g. infected gingival pockets.

For which purpose can you use the OXYSAFE Gel Professional?

OXYSAFE Gel Professional is used in the treatment of periodontitis and peri-implantitis.

How do you use OXYSAFE Gel Professional?

OXYSAFE Gel Professional is being used in dentistry in case of pockets with a minimum depth of 5 mm and peri-implantitis.

When must OXYSAFE Gel Professional not be used?

With the exception of titanium, OXYSAFE Gel Professional must not be used if metal implants or metal prosthesis are present. Make sure to remove any prosthesis prior to using OXYSAFE Gel Professional.

How do you use OXYSAFE Gel Professional?

Dosage

The dosage depends on the type of pocket and depth of the pocket. The maximum dose per treatment is 3 ml.

How to use OXYSAFE Gel Professional in dentistry in case of pockets with a minimum depth of 5 mm.

- Place a plastic needle under the pocket very slowly and push the OXYSAFE Gel Professional out of the syringe until the OXYSAFE Gel Professional which comes out of the pocket looks clean. Just after the treatment of the pocket, the patient might experience a short moment of a warm and/or painful feeling, which should disappear immediately.
- Leave the OXYSAFE Gel Professional in situ for 5 minutes.
- Clean the pocket by rinsing it with saline solution.
- Place a plastic needle under the pocket very slowly and push the OXYSAFE Gel Professional out of the syringe until the OXYSAFE Gel Professional comes out of the pocket and let the OXYSAFE Gel Professional remain in the pocket.
- In some exceptional cases blockage of the application tip can occur when using capillary tips with an ultra-small bore at the end. When blockage occurs use a new or alternative tip with a wider bore.

When do you have to take extra care with OXYSAFE Gel Professional?

Possible reactions

If you want to use the OXYSAFE Gel Professional and in case of an inflammation at that spot, the patient can feel pain, which should disappear fast. The pain also can be felt in the head. All these symptoms are the result of a locally increase in pressure due to the OXYSAFE Gel Professional. Also be careful when treating a furcation.

What should you do if the pain does not disappear?

Fill a syringe with a saline solution and rinse the pocket until the pain disappears.

How to use in dentistry in case of peri-implantitis.

Place a plastic needle under the space around the implant very slowly and push the OXYSAFE Gel Professional out of the syringe until the OXYSAFE Gel Professional which comes out of the pocket looks clean.

What to consider in pregnancy.

As far as is known it is safe for the unborn child if OXYSAFE Gel Professional is used during pregnancy and/or during lactation. Generally, as little medication as possible should be used during the first three months of pregnancy. OXYSAFE Gel Professional is not a drug. However, it is recommended to restrict its use during this period.“

After periodontal surgery during which the OXYSAFE Gel Professional has been used, the patient should continue with OXYSAFE Liquid (REF: HW-155042) according to the manufacturer’s instructions.

Additional information OXYSAFE Gel Professional.

The critical ingredient in the OXYSAFE Gel Professional is the Ardoz-X™ technology; based on an oxygen-releasing complex

Appearance and packaging of the OXYSAFE Gel Professional .

OXYSAFE Gel Professional is a opaque, white Gel.

OXYSAFE Gel Professional is packed in plastic syringes with a content of 1 ml. One outer packaging contains 3 syringes. Due to the fact that OXYSAFE Gel Professional reacts with metals, metal needles can not be used to apply the OXYSAFE Gel Professional.

This user information was last revised 11-2017

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