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Instructions of use O7 active® PRO mouthwash

What is O7 active® PRO mouthwash?

O7 active® PRO mouthwash is a product with a patented active oxygen technology. This technology becomes active at the moment of contact with the skin or mucous membranes.

How does the O7 active® PRO mouthwash work?

O7 active® PRO mouthwash contains the patented Ardox-X® technology which becomes active at the moment of contact with the skin or mucous membranes. The technology is based on a stabilized hydrocarbon-oxoborate complex that generates a non-radical form of oxygen under the influence of saliva. The hydrocarbon-oxoborate complex functions as a matrix that releases active oxygen on the area that has to be treated. The oxygen-enriched micro-environment has a proven positive effect on the (infected) wound treatment, e.g. infected pockets.

For what purpose can the O7 active® PRO mouthwash be used?

O7 active® PRO mouthwash is used for the treatment of gingivitis, periodontitis and peri-implantitis.

In which cases can the O7 active® PRO mouthwash be used?

O7 active® PRO mouthwash is used in treatment of gingivitis, periodontitis and peri-implantitis by a dentist.

In which case the O7 active® PRO mouthwash can't be used?

O7 active® PRO mouthwash cannot be used in combination with metal implants or metal prosthesis, with the exception of titanium. Before using O7 active® PRO mouthwash, first remove the prosthesis.

How is the O7 active® PRO mouthwash used?

Dosage

Rinse and/or brush the mouth three times a day during 1 minute with approximately 10 ml of O7 active® PRO mouthwash and spit out after use. Don't rinse the mouth with water afterwards.

Contamination prevention

To prevent contamination always use the provided measuring cup and rinse the cup after use with tap water.

How long can the O7 active® PRO mouthwash be used?

Treatment can be stopped when the patient is cured of the infection, but no longer than 7 years.

What should you pay attention to during a pregnancy?

O7 active® PRO mouthwash may, to the best of our knowledge, be used without risk to the embryo during pregnancy and / or lactation, according to the instructions. In general, you should use as few medicines as possible during the first trimester of pregnancy. O7 active® PRO mouthwash is not a medicine. Nevertheless, restraint with the use during this period is still advised.

Additional information O7 active® PRO mouthwash.

The critical ingredient in the O7 active® PRO mouthwash is the Ardox-X® technology; based on an oxygen donating complex.

Appearance and packaging of the O7 active pro mouthwash?

Appearance: O7 active® PRO mouthwash is a clear to slight opaque colourless liquid.

Packaging: O7 active® PRO mouthwash is packed in a white bottle with a content of 250 ml.

These instructions for use are last revised 02-2018.

Distributed by: Sunstar Oral Company Int. B.V.
Hogeweyerslaan 221
1382 JL Weesp
The Netherlands